


# SPRINT® Air Ankle Walker

Patent Pending

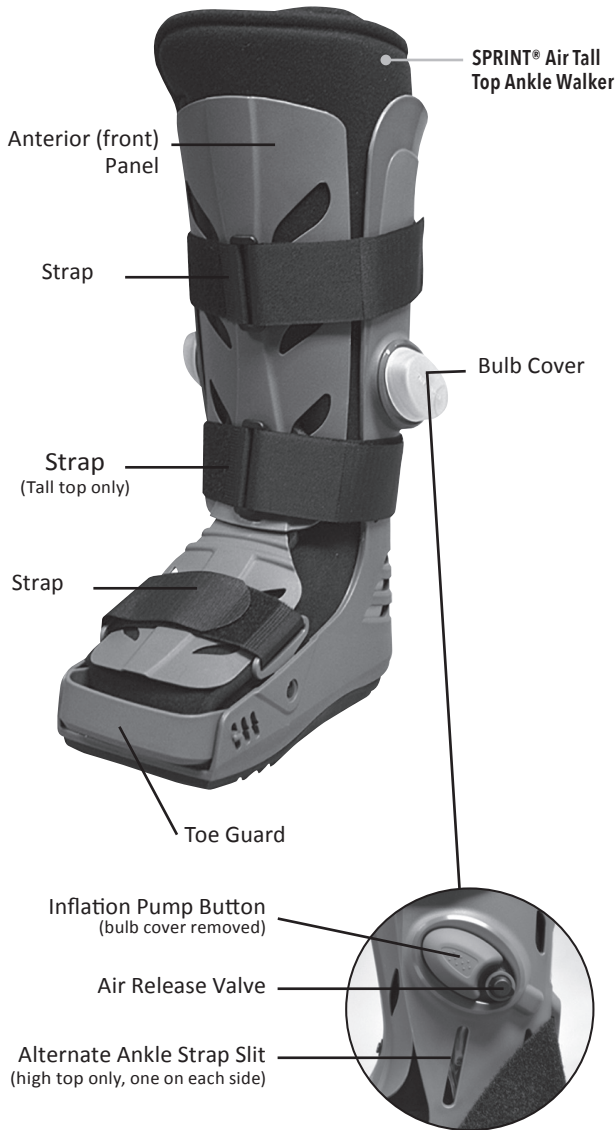

Online application instructions.



NOT MADE WITH natural rubber latex

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function and warranty of the device.

**This device is intended for single patient use.**



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

## APPLICATION INSTRUCTIONS

**NOTE:** High pressure in the bladders provides more support when walking; lower pressure is more comfortable when sitting or reclining. Pressure can be adjusted by inflating/deflating the air bladders or tightening/loosening the device straps.

### Brace Application

1. Unfasten straps, remove anterior panel, open liner and remove bulb covers found on either side of the device.
2. While seated, place leg in brace and position heel at the back of the walker. Wrap liner around the leg and foot and secure.
3. Replace the anterior panel.
4. Beginning with the toe strap, thread straps through links and work up the leg by securing each strap. For the tall walker, strap can be relocated through the alternate ankle strap slit, beneath the air release valve, for a more custom fit.

**NOTE:** Tall ankle walker has two leg straps and the low top ankle walker has one leg strap.



**WARNING:** Do not over-inflate the air bladders. Improper inflation may cause skin irritation and/or damage to the device.

5. To inflate the air bladders, pump the large inflation button on the sides of the device until air bladders feel snug.

### Brace Removal

1. When removing the device, deflate the air bladders until they are no longer snug. Deflating the air bladders prior to removing the device will ensure proper fit when reapplying. Follow the brace application instructions (above) when reapplying the brace.
2. To deflate the air bladders, open the air release valve cap (the smaller button next to the inflation pump button) and press the air release valve to release air.

### Toe Guard Adjustment

The Toe Guard has 3 settings and can be adjusted to accommodate foot length. Toe Guard may also be removed if desired.

1. To adjust Toe Guard, pull guard upward and then pull out from both sides of slots.
2. Re-position the guard into desired slot and push downward to lock the guard into place.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at 800-328-1095.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.