

BIRD & CRONIN INC.®

Improving lives with quality medical products

Rib Belt – Universal

NOTE: Instructions cover multiple product families, images may vary.

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

NOT MADE WITH natural rubber latex



Figure 1



Figure 2



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Wrap the rib belt around your back at the level of your lower rib cage.
2. Wrap the end of the belt without hook across your body and hold against ribs (Figure 1).
3. Stretch the other end over the belt and across your body in the opposite direction and secure the hook strip to the hook-receptive belt for the desired level of compression and support (Figure 2).

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.



Bird & Cronin, Inc
1200 Trapp Road · Eagan, MN USA 55121
800-328-1095 · www.birdcronin.com

©Bird & Cronin®, Inc. 10001386A