

Total Compression Knee Immobilizer - Universal

Designed for maximum compression for the knee and leg


APPLICATION INSTRUCTIONS

It is critical to determine the proper length of the knee immobilizer prior to the actual application. A correctly sized immobilizer fits on the leg 2" to 3" above the lateral malleolus and 6" to 8" below the femoral head.


Reorder No.	Size	Length
0814 2542	Universal	16"
0814 2544	Universal	20"
0814 2546	Universal	24"



1. Open "Total Compression Knee Immobilizer" - Universal.

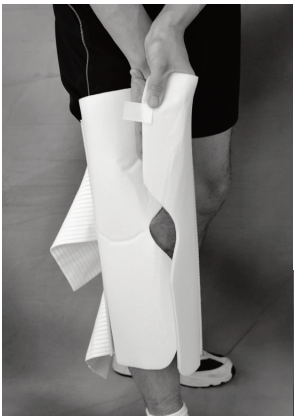

2. Place immobilizer on the leg so that posterior (rear) stays are positioned properly. The optional popliteal adhesive foam pad adds contour and minimal flexion to the compression immobilizer.

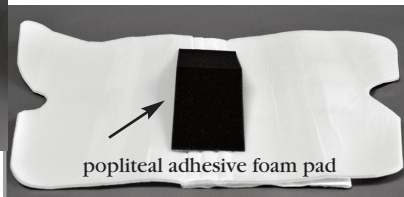

3. Position pressure-sensitive medial and lateral stays along the midline (center) of the leg.



4. Wrap the elastic compression straps to secure immobilizer.



5. Secure patellar strap over the patella.

Note: Number of straps vary depending on the length of immobilizer.

Warning-Caution: Carefully read enclosed instructions and warning prior to use. Correct application and sizing is vital to the proper functioning of the device. This device will not prevent or reduce all injuries. Patients with poor peripheral circulation, diabetes or decreased skin sensitivity should not use this device. Consult your medical professional immediately if you experience sensation changes, unusual reactions, swelling, or prolonged pain while using this device. The device is for single patient use only.

BIRD & CRONIN INC.

Quality Medical Products To Help Improve Your Life

1200 Trapp Road • Eagan, Minnesota USA 55121

800/328-1095 • Fax 800/279-7934

email sales@birdcronin.com • www.birdcronin.com

10002542