



## Buddy Splint


1

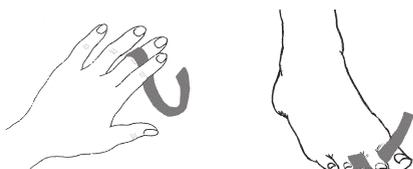

2

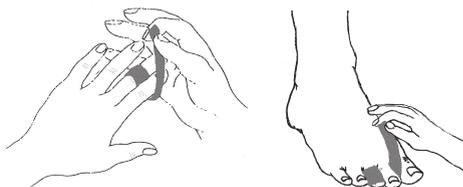

3

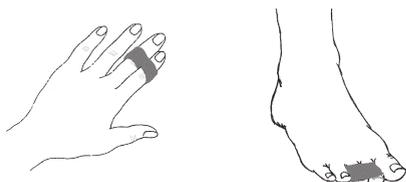

**NOT MADE WITH**  
natural rubber latex

Please review the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

### Intended for single patient use.



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Place loop around finger/toe next to the injured finger/toe.
2. Wrap strap around the injured finger/toe and secure.
3. Adjust for comfort.

### CARE INSTRUCTIONS

Hand wash with warm water and mild detergent. Allow to air dry.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.