

Tennis Elbow Support

NOTE: Instructions cover multiple product families, images may vary.

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.



Intended for single patient use.

NOT MADE WITH
natural rubber latex



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Position tennis elbow support below the elbow as shown above.
2. Guide end of strap through link, pull back and secure.
3. Re-adjust as needed for comfort.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

