

BIRD & CRONIN INC.®

Improving lives with quality medical products

Velpeau Immobilizer

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

NOT MADE WITH natural rubber latex



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Slide affected arm into sling pouch (A) and position elbow snugly into the posterior curve. Optional: Place thumb in thumb loop (B).
2. Wrap shoulder strap (C) diagonally across your back and over your unaffected shoulder.
3. Guide end of strap (C) through O-ring (D) on sling pouch (A), loop back and pull until pouch is at proper height. Secure end of strap.
4. Wrap strap (E) around waist like a belt.
5. Guide end of strap (E) through O-ring (F) on sling pouch (A), loop back and pull until fitted. Secure end of strap.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.



Bird & Cronin, Inc
1200 Trapp Road · Eagan, MN USA 55121
800-328-1095 · www.birdcronin.com