

# Tri-Panel Knee Immobilizer

NOTE: Instructions cover multiple product families, images may vary.



Figure 1

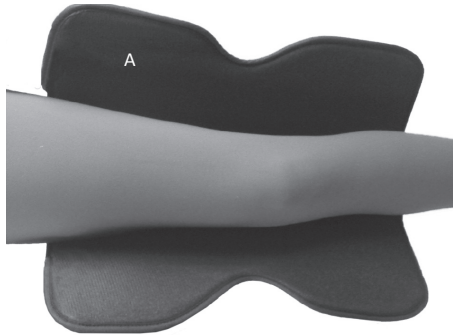


Figure 2



Figure 3

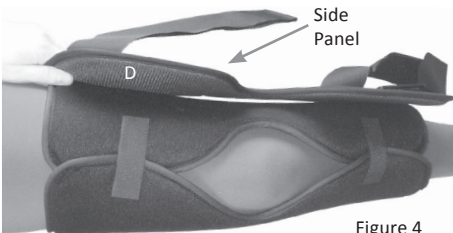


Figure 4



Figure 5

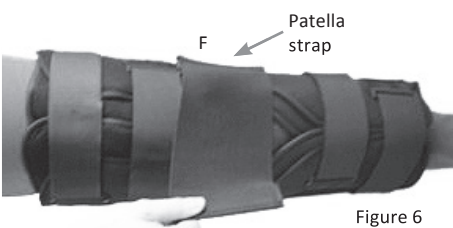


Figure 6

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

## Intended for single patient use.

**WARNING:** This product should be used only under the supervision of a medical professional. If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

**CAUTION:** It is critical to determine the required length prior to selecting a brace for application. A correctly sized knee immobilizer fits on the leg 2" to 3" above the lateral malleolus and 6" to 8" below the femoral head.

## APPLICATION INSTRUCTIONS

1. Sit on a flat surface with the affected leg in full extension.
2. Unfasten and unfold the knee immobilizer (A). Detach and set aside the two 3" hook strips (B) and the two side panels (D) (Figure 1).
3. Center immobilizer (A) under the leg and align the posterior stays (C) along the back of the leg (Figure 2).
4. Wrap immobilizer (A) around the leg and hold in place using the two 3" hook strips (B) (Figure 3).
5. Center a side panel (D) along the inside (medial) length of the leg and press to attach to the wrap. Center remaining side panel (D) along the outside (lateral) length and press to attach (Figure 4).
6. Wrap and fasten the loop-and-lock straps located directly above and below the knee (E). Wrap and fasten remaining straps (E) (Figure 5).
7. Center patella strap (F) at knee cap and attach one end to the brace. Pull strap to tighten and secure opposite end to brace. (Patella strap is optional, use when required) (Figure 6).

**NOTE:** Use loop and lock feature on the leg straps to adjust fit and compression.

**NOTE:** Number of straps varies depending on the length of immobilizer.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product and warranty information or contact a customer care representative at **800-328-1095**.

**NOTE:** This device is only one element in the overall treatment program administered by a medical professional. While every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

**NOT MADE WITH  
natural rubber latex**