

Universal Wrist Thumb Spica



Figure 1



Figure 2

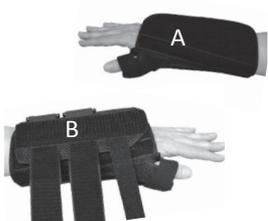


Figure 3



Figure 4



Figure 5

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Separate brace into two pieces, thumb spica component (A) and dorsal component (B) (Figure 1).
2. Position thumb spica component (A) with smooth side facing up. Slide thumb of affected hand into thumb spica. Position curved stay in the palm of the hand. (Figure 2).
3. Align dorsal stay (B) component with middle knuckle and place around the thumb spica (A). Adjust for wrist circumference by moving component medially or laterally (Figure 3).
4. Pull and wrap straps around the wrist. Secure each strap when desired fit and level of compression is reached (Figure 4).
6. Wrap thumb strap around thumb and secure (Figure 5).
7. Adjust straps as needed when swelling of the hand, wrist or thumb changes.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH
natural rubber latex**