

## Wrist Brace

**NOTE:** Instructions cover multiple product families, images may vary.

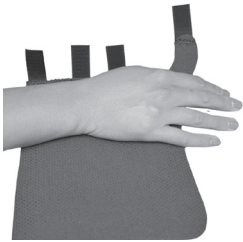


Figure 1



Figure 2



Figure 3



Figure 4

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

### Intended for single patient use.



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Unfasten straps and place affected hand palm down into brace. (Figure 1)
2. Wrap brace over top of hand.
3. Secure the middle hook straps to the top of the brace. (Figure 2)
4. Place thumb strap between thumb and index finger. Fasten the hook to the top of brace. (Figure 3)
5. Adjust straps as needed when swelling of the hand, wrist or thumb changes.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**