


## Arm Sling with Foam Strap

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

**Intended for single patient use.**

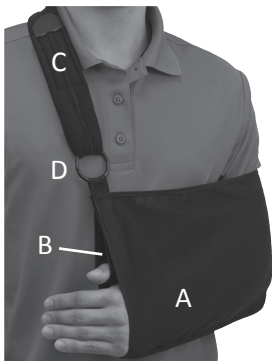

**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Slide affected arm into sling pouch (A) and position elbow snugly into the back corner. Optional: Place thumb in thumb loop (B).
2. Wrap strap (C) diagonally across your back and over your unaffected shoulder.
3. Guide end of strap (C) through O-ring (D) on sling pouch (A), loop back and secure back onto itself.
4. Adjust for proper height and comfort

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**

