

# BIRD & CRONIN<sup>®</sup>

a division of  Dynatronics

Improving lives with quality medical products

## Spiffys<sup>®</sup> Modesty Shorts



Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.



**WARNING:** If you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

#### Acute care application:

1. Disengage hook and loop seams on either side for application.
2. With shorts open flat, the back of the shorts can be placed under the patient.
3. The front of the shorts can be drawn up under the patient.
4. Align the seams on each side and engage the matching colored hook and loop.

#### Non-acute care applications:

Shorts may be applied in a normal fashion.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**



Bird & Cronin LLC  
1200 Trapp Road · Eagan, MN USA 55121  
800-328-1095 · [www.birdcronin.com](http://www.birdcronin.com)