

Solo Universal Wrist Brace

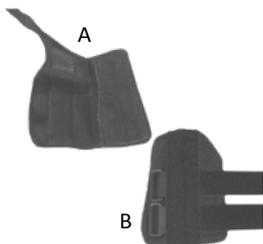


Figure 1



Figure 2



Figure 3



Figure 4

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Unfasten straps and separate brace into two pieces. Position component (A) flat with sew-in label facing down. (Figure 1)
2. Position affected hand into component (A). The curved stay should be positioned at the crease of the palm. (Figure 2)
- 3 Place component (B) on dorsal (top) side of hand. Attach component (B) to component (A) using hook and loop. Links should be directed towards medial aspect of wrist to align with the thumb side of the wrist.
4. Thread wrist straps through links securing each strap until desired level of fit and compression is reached. (Figure 3)
5. Place thumb strap between thumb and index finger. Fasten the hook to the top of brace. (Figure 4)
6. Adjust straps as needed when swelling of the hand, wrist or thumb changes.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH
natural rubber latex**

