


Deluxe Telemetry Pouch

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

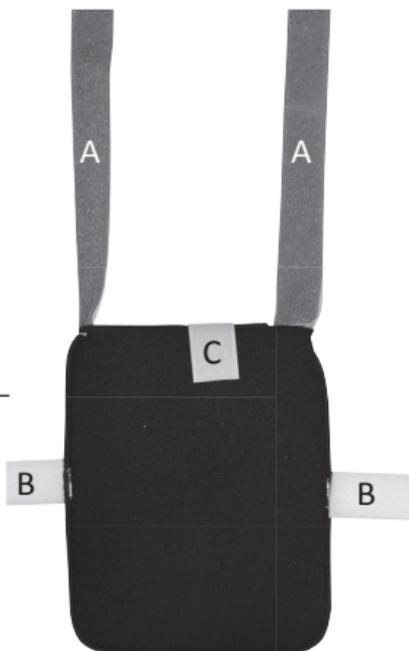

APPLICATION INSTRUCTIONS

1. Position pouch on patient at desired location.
2. Place the top strap (A) over the opposite shoulder, across the back, and attach to pouch with hook and loop closure tab.
3. Wrap circumferential strap (B) around waist and attach to pouch with hook and loop closure.

NOTE: Straps can be trimmed to desired length.

4. Lift pouch security tab (C) and place telemetry unit inside pouch with lead wires extending from top of pouch. Secure tab to pouch.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH
natural rubber latex**

