

## Boxer Splint



Figure 1



Figure 2



Figure 3



Figure 4



Figure 5



Figure 6

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

### Intended for single patient use.



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Open straps and place splint on a flat surface with opening facing up. (Figure 1)
2. Rest hand in splint. (Figure 2)
3. Place the 4th and 5th fingers in the top portion of the splint and secure strap. (Figure 3)
4. Secure bottom strap around wrist. (Figure 4)
5. Slide middle strap through plastic ring and secure. (Figure 5)
6. Place palmer pad where most comfortable.
7. Re-adjust if necessary. (Figure 6)

**NOTE:** Wash plastic shell with mild soap and a damp cloth. Wipe dry.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**