

## Sacroiliac Belt and Stabilization Pad

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

**Intended for single patient use.**



**NOTE:** Sacroiliac belt provides support that may alleviate pain, reduce inflammation and restore function. Designed to stabilize the pelvis and prevent excessive movement of the SI joint.

### APPLICATION INSTRUCTIONS

1. Securely fasten the belt around the pelvis by threading the belt through metal “D” ring and fasten it with the hook and loop.
2. The belt should be positioned approximately two inches below the top of the hip bones. Position of the belt is important for its effectiveness.
3. For additional stabilization support or for patients who have a natural depression over the sacrum, place the pad over the sacrum as shown.
4. Adjust the tension of the Sacroiliac Belt for desired comfort and support.

**NOTE:** It is recommended that the belt be worn over the patient’s undergarments.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**

