


FlexiGrip – Tubular Elastic Support Bandage



Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

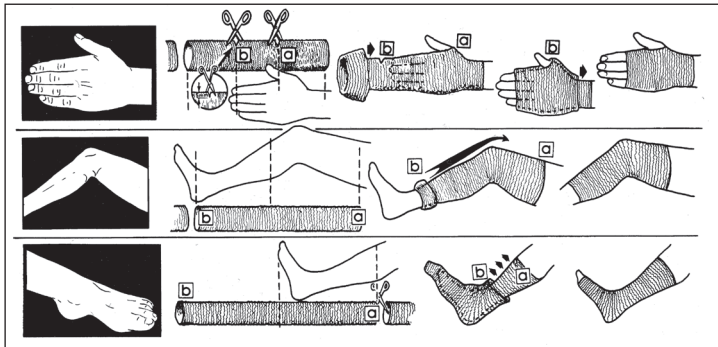

WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Cut a length of FlexiGrip that corresponds to the area that will be covered.
2. Draw the bandage evenly over the limb as a sleeve.

NOTE: If more compression is recommended, cut double the length and fold material back over the area as a second layer.

Size	Width	Fits Circumference	Description
A	2.00"	5" – 7"	Infant foot or arm
B	2.50"	7.5" – 8"	Small hand and arm
C	2.75"	8.5" – 8.75"	Medium arm, Small ankle
D	3.00"	9" – 10"	Large arm, Medium ankle, Small knee
E	3.50"	10.5" – 11.5"	Large ankle, Medium knee, Small thigh
F	4.00"	12" – 14"	Large knee, Medium thigh
G	5.00"	15" – 20"	Large thigh
J	6.00"	18" – 24"	Small trunk
K	8.00"	24" – 32"	Medium trunk
L	12.00"	30" – 40"	Large trunk

Please visit our website at www.birdcronin.com for additional product and warranty information or contact a customer care representative at **800-328-1095**.

NOTE: This device is only one element in the overall treatment program administered by a medical professional. While every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

**NOT MADE WITH
natural rubber latex**