

Bicro™ Shoulder Immobilizer



Figure 1



Figure 2



Figure 3

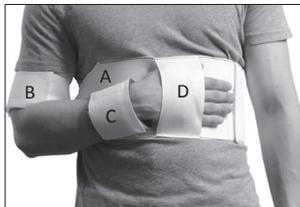


Figure 4

**NOT MADE WITH
natural rubber latex**

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Unfold swathe (A) and position around front mid-section. Wrap swathe around waist and secure. (Figure 1)
2. Wrap humeral strap (B) around affected arm and secure with hook and loop closure. (Figure 2)
3. Place wrist of affected arm in foam padded wrist cuff (C) and secure with hook and loop closure. (Figure 3).
4. Slide hand of affected arm through hand cuff (D) to avoid wrist drop. (Figure 4)

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product and warranty information or contact a customer care representative at **800-328-1095**.

NOTICE: This device is only one element in the overall treatment program administered by a medical professional. While every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.