

Genu-Lite™ Knee Support

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

NOT MADE WITH natural rubber latex



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Open sleeve and pull onto leg.
2. Ensure silicone ring of the patella opening is around the knee cap.

NOTE: Contact with ointment, oil and excessive perspiration may shorten the useful life of the support by damaging the elastic fibers.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at **www.birdcronin.com** for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

