

Improving lives with quality medical products

L'TIMATE[®] Knee Wrap



Figure 1 Outside of Wrap



Figure 2 Inside of Wrap



Figure 3



Figure 4



Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

- 1. Open sleeve and position the buttress on the side of the patella where tracking support is desired.
- 2. Ensure patella opening is centered over the kneecap. (Figure 3)
- 3. Once sleeve is positioned as desired, wrap lateral pull straps firmly around the knee engaging the hook closures for comfortable support. (Figure 3)
- 4. Adjust straps to achieve desired fit and level of compression.

NOTE: The position of the U-shaped patella buttress can be modified by pulling edges through the slits of the sleeve and securing the hook to the wrap.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at **www.birdcronin.com** for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.



Bird & Cronin LLC 1200 Trapp Road · Eagan, MN USA 55121 800-328-1095 · www.birdcronin.com