

**TRACH-MATE™**  
**Tracheostomy Tube Holder**

**PROPER  
APPLICATION ILLUSTRATIONS**



**APPLICATION INSTRUCTIONS**

1. Position and secure neckband around the patients neck.
2. Thread the narrow fastener tabs through the opening on the tracheostomy tube and back over the flanges securing the hook and loop together.
3. Secure the tracheostomy tube in place by adjusting the length of the neckband.
4. Cut off excess material.

***BIRD & CRONIN INC.***

*Designers and Manufacturers of Quality Medical Products*  
1200 Trapp Road • Eagan, Minnesota USA 55121

**TRACH-MATE™**  
**Tracheostomy Tube Holder**

**PROPER  
APPLICATION ILLUSTRATIONS**



**APPLICATION INSTRUCTIONS**

1. Position and secure neckband around the patients neck.
2. Thread the narrow fastener tabs through the opening on the tracheostomy tube and back over the flanges securing the hook and loop together.
3. Secure the tracheostomy tube in place by adjusting the length of the neckband.
4. Cut off excess material.

***BIRD & CRONIN INC.***

*Designers and Manufacturers of Quality Medical Products*  
1200 Trapp Road • Eagan, Minnesota USA 55121

3" X 6"